

1. Manufacture a funnel.

Identify a sheet metal manufacturer to make the funnel according to the design drawings below.


2. Select a barrel and identify an appropriate location.

Barrels should be plastic, as metal barrels could rust over time. Health facility supervisors and waste handlers should decide together on a site for the barrel. It should be dry, secure, and convenient.

3. Attach the funnel to the sharps barrel.

Screw the metal funnel into the hole on top of the barrel. Rubber cement or other sealant may be used to help secure connection if needed.

Metal funnel

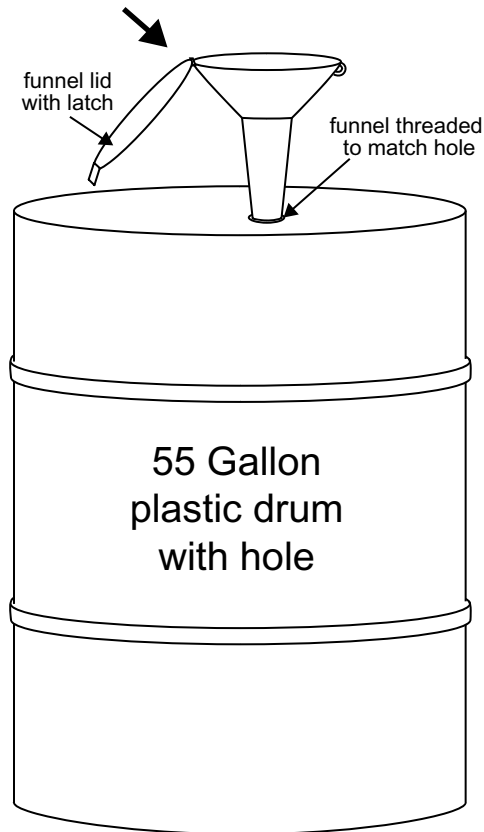

Funnel details

